


# GUTTA FILL® STEP-BY-STEP INSTRUCTIONS

## INDICATIONS FOR USE

Thermal obturators used in root canal therapy by dentists.  
Carrier-based obturators coated with flowable gutta percha that are heated and used to fill constant or variable taper, 04, 06, 08, or 10 endodontic canals.

## CONTRAINDICATIONS

None known.

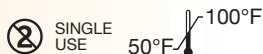

## NOTICE

This product contains no known elements which may cause allergic reactions.

## PRECAUTIONS

1. US Federal Law restricts this device for sale by or on the order of a dentist.
2. Dentists and staff should utilize appropriate handling measures and protective gear when using this product.

## ADVERSE REACTIONS

None known

1. Choose the **Guttafill®** obturator that matches the largest file taken to length. For example, if you finish the canal with a Size 20 file with taper 04, 06, 08, or 10, then fill with the Size 20 **Guttafill®** obturator.
2. **Guttafill®** can be used for any tapered canal.
3. Place the **Guttafill®** obturator in a 5.25% sodium hypochlorite solution for one minute, then rinse with 70% alcohol and allow carrier to dry thoroughly before use.
4. **Guttafill®** can be used with the same heat setting in your current oven.
5. Dry the canal and apply a thin layer of sealer.
6. Set the stopper to the working length (or use the calibration rings on the **GuttaFill®** obturator)
7. Set the oven to the corresponding ISO tip size.
8. Insert the **Guttafill®** obturator into the oven and start the heating cycle.
9. Remove the obturator when the oven beeps.
10. Immediately place the obturator smoothly in the canal to the working length.
11. Cut the obturator off at the orifice level by pushing against it with a spoon excavator.

Manufactured by: US Endodontics, LLC  
309 Wesley Street Johnson City, TN 37601  
Tel: 423-283-0047

INTRODUCING

# GUTTA FILL

## The World's First Thermal Obturator with a Pink Gutta Percha Carrier

**GuttaFill®** Vulcanized Compounding produces the world's first pink gutta percha carrier that gives you all the benefits you want:

- Easy to use
- Use your current obturator oven at the same settings
- Fills all canal shapes
- Easy to place
- Easy to make post space
- Easy to retreat
- Easy to get great 3D fills for all canal types
- Up to 1/2 the cost of GuttaCore®

